



Masimo SafetyNetTM Radius T^o

Setup Instructions

Warning: Do not self-diagnose or self-medicate on the basis of the measurements. Always consult your doctor.

GET TO KNOW YOUR SYSTEM




Masimo SafetyNet App

Masimo SafetyNet, collects and displays continuous data from paired wearable sensor. It also sends data to your care provider.

Radius T^o Sensor


Worn on your chest, the sensor measures body temperature and sends your data to the Masimo SafetyNet App.

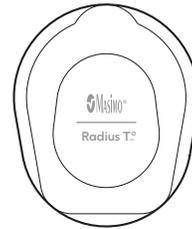

Getting Started


1 Set-up Masimo SafetyNet App

Follow instructions from the email or text message invitation to download the Masimo SafetyNet App and create an account.

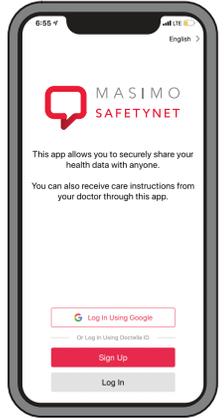


2 Apply Sensor


Before applying, make sure the application site on the chest is hair-free, clean and dry.

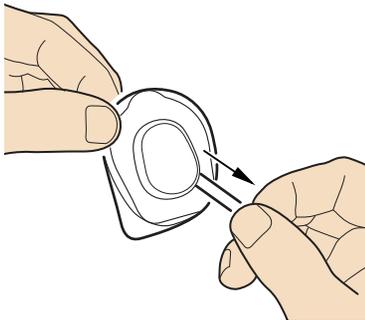

Remove sensor from pouch and pull tab to activate battery.

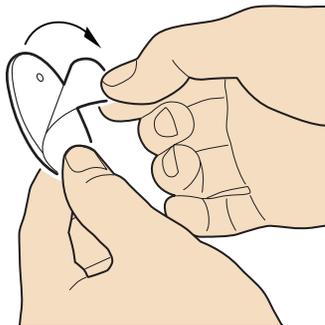


Remove plastic film from the back of the sensor.

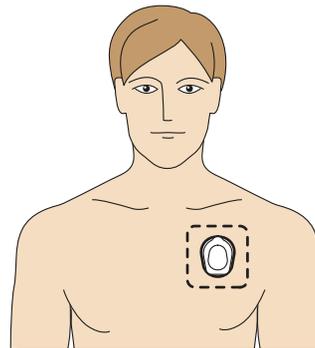

Apply adhesive sensor on the upper chest below the left collar bone as shown in diagram.

3 Pair Sensor

Under Masimo SafetyNet App **Settings**, enter the **Add Device** menu, and follow instructions for pairing the Radius T^o sensor to your phone.

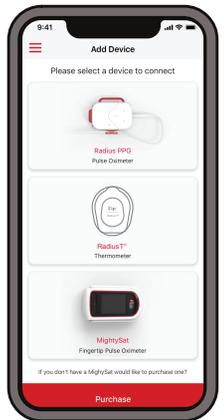


For additional resources, including videos and instructions, visit mymasimo.com

© 2020 Masimo. All rights reserved. 301644/81088-0720

Daily Tasks on Phone


Example shown – Your screens may differ

Begin CareProgram™

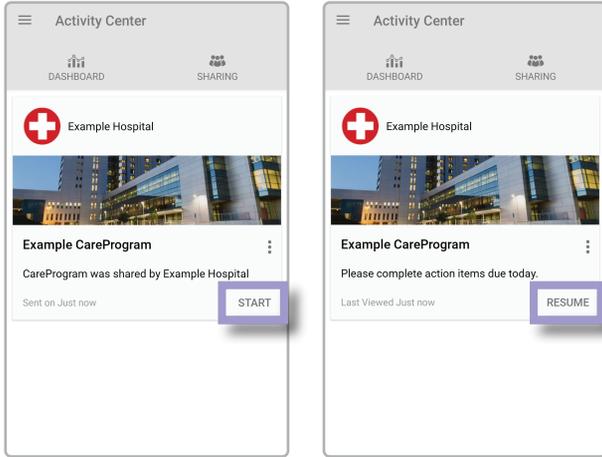

Select "Start" to begin your CareProgram or "Resume" to continue if you started it on a previous day.


Answer Questions

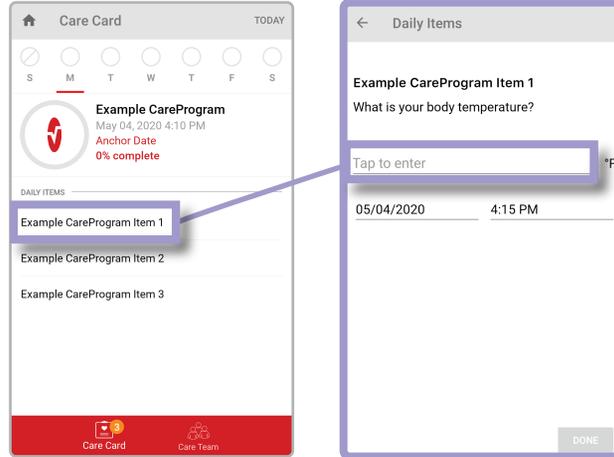

Select each item listed under "Daily Items" to answer question(s) from your healthcare provider.

Answer the question that pops up. Repeat until you've completed all "Daily Items" for each day.

Check Connection

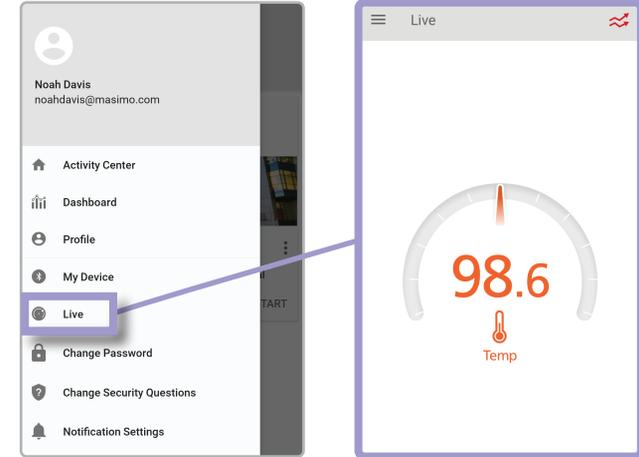


From Activity Center, tap ≡ icon to open the menu. Tap "Live" to view your live data from your sensor.

If your sensor is applied correctly and connected, you will see your live data with number.

Caring for Your Sensor



If battery light is orange or red, this means that the battery is close to depletion.

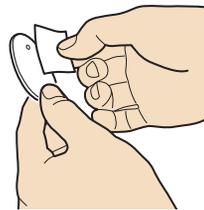

If needed, to remove and reapply the sensor, gently wipe the exposed adhesive with an alcohol swab and allow to dry before reapplying.



Sensor can be worn in the shower but should not be submerged.